


Ipas MVA Plus[®]



Ipas Manual Vacuum Aspirator Plus[®] (MVA) is a safe, effective and affordable tool for removing uterine content up to 12 weeks gestation. It is a preferred method for uterine evacuation recommended by World Health Organization (WHO) and International Federation of Gynecology and Obstetrics (FIGO).

1 Ipas MVA Plus Kit = 1 Aspirator + 8 Easy Grip colour coded cannulae:
sizes (4, 5, 6, 7, 8, 9, 10 & 12 mm)

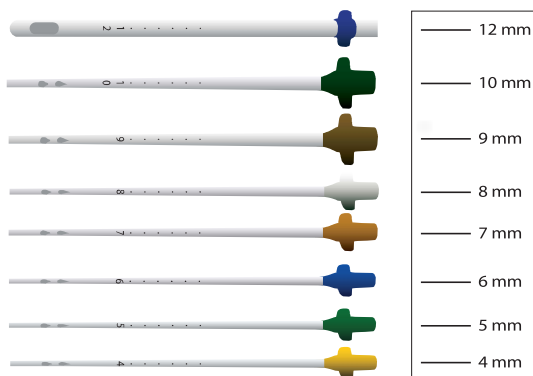

Selecting the appropriate cannulae size

Gestational age	Suggested cannulae size
4 to 6 weeks	4 – 7 mm
7 to 9 weeks	5 – 10 mm
9 to 12 weeks	8 – 12 mm

How to use

The cannula is inserted into the uterus and attached to the aspirator. The healthcare provider creates a vacuum by pulling the plunger. When the buttons on the aspirator are released, it pulls the contents of the uterus through the cannula into the chamber of the aspirator.

Indications for use

Uterine evacuation

- In the first trimester following early pregnancy loss due to induced, spontaneous, incomplete or missed abortion/miscarriage
- After vaginal delivery

Uterine sampling

- For endometrial biopsy
- In abnormal uterine bleeding or screening for endometrial infection

Benefits

- Double valve buttons for greater pressure
- Affordable and inexpensive
- Extremely safe and 98 - 100% effective
- Easy to disassemble and reassemble
- Portable and does not require electricity
- Does not require general anaesthesia
- Less than 1% chance of any complication
- Offers complete evacuation of the uterus
- Can be used for outpatient care
- Only aspirator listed by the US FDA to withstand reprocessing for up to 25 times

When to replace

- After 25 uses
- The cylinder or cannula is brittle
- Presence of tissue deposits
- The aspirator or cannula is broken
- The aspirator no longer holds vacuum

...your life saving tool

Packing

1 complete kit contains Ipas MVA Plus[®] Aspirator + 8 EasyGrip[®] cannula + silicone gel lubricant